

Master - Technical Bulletin

Product:	VISIONFIT SC
Date:	2019/11/19 (November 19, 2019)
Bulletin number and title:	26 – New software release 1.8
Issue has a relevance for the device safety:	NO
Field Safety Corrective Action:	Not required
Issue has a relevance for the device performance:	YES
Reason:	<ul style="list-style-type: none"> - New software version - New operating manual
S/N affected:	All
Scope of notice:	All Distributors/Other
Service time:	Less than 5 minutes for software update procedure
Service parts:	APK_SC_RELOAD_V1_8_Official.apk
Description:	See next pages
Required action:	Update recommended
Hardware compatibility:	All
For further info please contact:	<p>Adaptica srl Via San Marco, 9/h – 35129, Padova(PD), Italy Tel: +39 049 773 968 Fax: +39 049 097 0901 Email: support@adaptica.com</p>

Summary of changes and bugs fixed in SW version 1.8

- Objective refraction measurement data import through Bluetooth communication. The user can select the type of 2WIN Connection: WiFi or Bluetooth. If 2WIN Connection is set to Bluetooth, 2WIN's measurement can be directly imported from VisionFit SC application Reference tab.
- AQUID Chart optotypes remote control from VisionFit SC application. Possibility to set the measured Visual Acuity to VisionFit SC report.
- Custom Profiles creation and management. Profiles allow the user to save a set of custom settings which can be retrieved in any moment.
- New operator manual