

KALEIDOS plus instructions for CR occluder positioning

KALEIDOS plus includes Corneal Reflexes function, which is a guided procedure for the detection of possible phorias and tropias (please refer to 2WIN Operator Manual for further instructions of this function).


A complete Corneal Reflexes exam is made of three measurements:

- One binocular measurement
- One monocular measurement covering left eye
- One monocular measurement covering right eye

To cover the patient's eye in the two monocular measurements Adaptica IR occluder is needed.

Guide the patient with these instructions for the correct positioning of the IR occluder:

1. Hold the occluder in a vertical position and with the handle downwards
2. Hold the occluder using the ipsilateral hand to the covered eye (for example: hold the occluder with the right hand if you need to cover the right eye, and vice versa)
3. Place the occluder in front of the eye, keeping the edge of the occluder in contact with the side of the nose
4. Do not press the occluder against the eye: eyelid movements shall be possible and comfortable
5. Look into KALEIDOS plus keeping the occluder in the correct position. Pay attention to fit KALEIDOS plus aperture: ambient light must not enter the device during the exam.

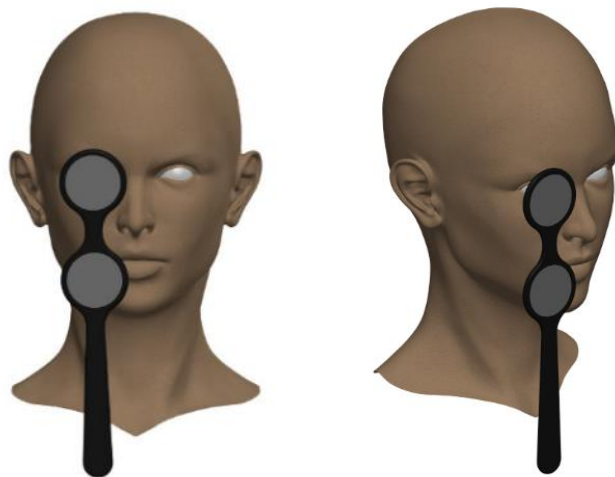

Correct occluder positioning